
Code	8475	Matr		Rev	20151126
------	-------------	------	--	-----	----------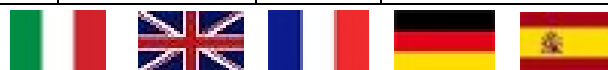

	IT	Sublimatore Professionale completo	2
	EN	Hot air Oxalic Acid Vaporizer	4
	FR	Sublimateur à acide oxalique	6
	DE	Heiß Luft-Verdampfer für Oxal säure	8
	ES	Sublimador profesional completo	10

Descrizione

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Description

Electric vaporizer for Varroa control treatment based on the use of oxalic acid brought to its gaseous state.

Preliminary operations before use

Prepare the vaporizer and heater.

Wear a face shield covering your nose and mouth (**protective mask type UP3**), goggles, gloves and coveralls made from acid-resistant rubber.

Safety directions and precaution

Pure oxalic acid is a noxious, toxic and corrosive substance which, however, is not dangerous if handled with the same level of care adopted for any other substance used for Varroa destructor control.

It becomes dangerous if oxalic acid powder is inhaled during manipulation.

For this reason, it is necessary to wear a good quality mask, protective goggles and good quality rubber gloves.

By doing this, you will also prevent any (however small) risk connected with minor acid leaks from the beehive.

During treatment, beekeepers should be aware of the wind direction and never work downwind. The amount of oxalic acid that could leak out is only a minimal fraction of the amount applied to the frames.

Using only the strictly necessary amounts of oxalic acid allows the treatment to be repeated more frequently without affecting egg-laying.

Characteristics

With a single vaporizer, 60 to 80 hives can be treated in one hour; every beehive will receive the same amount of oxalic acid (approximately 0.25 grams/hive) at the same temperature, without any toxic effect for the bees and ensuring extremely effective Varroa control.

Oxalic acid is introduced from below, under pressure and at constant temperature; in only a few seconds, the colder internal air is caused to flow out and the whole beehive is saturated - creating smoke flues wherever small cracks or holes are found.

Acid is vaporized by the hot air fed in at high rate (approximately 0.5 cubic m per minute).

Operating Instructions

- 3 Ensure that the electric system is in good condition.
- 4 Wear the recommended protective gear.
- 3 Screw out the glass cover and fill the tank contained inside the device body with the required amount of oxalic acid (max quantity shall not exceed 3/4 of the container capacity) then screw the cover back.

Only use crystal and not liquid oxalic acid

- 4 Pressure connect the heater (absorption: about 2200 W) to the vaporizer and to a 220 V power supply (a power generator with at least 3 kW power can also be used) –through a current stabilizer.
- 5 Switch on the heater and set it to 3, then wait until it reaches a temperature of 280°C (the oxalic acid begins to sublime after 4-5 minutes). Wait for a few minutes then select the required working temperature (approx. 200°C).

- 6 Insert the vaporizer nozzle into the beehive by lifting the special door; leave it in for approximately 15 seconds; it is not necessary to close the door at treatment end.

ALWAYS KEEP THE VAPORIZER VERTICAL

- 7 After work completion, turn the heater off and unplug it from the power supply; clean the vaporizer from any residual acid.

Caution: Do not leave the vaporizer in a closed environment before it has completely cooled off and has been thoroughly cleaned.

Caution: The vaporizer should not enter into contact with very damp air or liquids. This would affect both the resistor and the heater motor.

The guarantee shall not cover any damages caused by neglect or device misuse.

Treatment schedule

1-2 treatments for swarms or non-egg laying families.

Up to 4 treatments at one week's intervals for egg-laying families.

Please remember that oxalic acid will not reach into egg-laying cells with cappings.

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The machinery is guaranteed 24 MONTHS starting from the date of sale.

The guarantee is only valid if, when the machine is collected by our customer care or technical service staff, the owner can produce proof of purchase in the form of a fiscal receipt or invoice.

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The guarantee includes free-of-charge repairing and replacement of any part of the machinery that is found to have manufacturing or material defects by the manufacturer or the manufacturer's authorised person. This guarantee shall not apply to damages caused by negligence, misuse or use not in compliance with the directions contained in the instruction manual, as well as in case of accidents, alteration, tampering, wrong repairing or repairing with non-original parts, repairing by persons not authorised by Lega s.r.l. and damages during transport to/from the purchaser's. All electric parts (electric motors, controls etc.) and parts exposed to normal wear and tear as well as aesthetic parts are also not covered by the guarantee. All labour, packing, forwarding and transport charges shall be borne by the purchaser. Any defective parts which have been replaced shall be retained by and become the property of LEGA S.R.L. Any breakdown or defect which should occur during the guarantee period or after its last date shall not in any case entitle the purchaser to suspend the payments nor to any discount off the price of the machine. In any case, Lega s.r.l. shall not be held responsible for any damages resulting from the incorrect use of the machinery.

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