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INSTRUCTION MANUAL

Furetto

Dati tecnici

4

Technical data

6

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8

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10

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12

8480

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1.0 GENERAL SAFETY DIRECTIONS

The device includes dangerous parts which due to the presence of gas can cause serious damages to persons or property in case of: incorrect use, removal of guards or disconnection of safety devices, poor inspection and servicing.

The manufacturer may not be held responsible for failures, breaks or accidents resulting from incorrect use of the machine or failure to follow the directions contained in this manual.

1.1 IDENTIFICATION OF THE MACHINE

"Furetto", gas heated vaporiser against varroa.

IMPORTANT! DO NOT USE WITH OXALIC ACID

1.2 MACHINE OPERATORS

WARNING! FOR SAFETY PURPOSES, THIS MACHINE SHOULD BE USED BY SKILLED STAFF ONLY, AWARE OF THE INSTRUCTIONS CONTAINED IN THIS MANUAL WHICH FORMS AN INTEGRAL PART OF THE MACHINE.

"Furetto" has been designed for use by a single operator.

Basic technical and mechanical skills are required to correctly operate the machine and carry out maintenance and basic servicing.

2.0 OPERATING INSTRUCTIONS

Pressing and releasing button (5) moves the product to the serpentine where it immediately vaporises out of the spray nozzle (3) and enters the hive. Each spraying takes no more than 2 seconds.

The humidity of vapour depends on pumping frequency; it is recommended to repeat the press-and-release action at intervals of 5 seconds.

Before inserting the butane cartridge in the device, make sure the regulating valve (8) is perfectly closed; unscrew and take out the red container (9), place the cartridge inside it and then tighten it back in place.

IMPORTANT!
Keep away from flames or other heat sources when inserting the butane cartridge. Replace the butane cartridge only when it is completely empty. Furetto can be operated using 190 gr butane gas cartridges.

Fill the yellow container (11) with the liquid which is to be used for the treatment (making sure to filter it, if impurities are present) then tighten the container to the handle.
Place Furetto on a flat surface, light a match and hold it near the metal grid (4), open the gas valve (8) acting on the regulating valve.
The flame should start immediately. Allow the vaporiser to pre-heat for approximately 2 minutes. Start the treatment.

WARNING!
The metal grid gets very hot when the device is working: be very careful not to burn yourself!

When vapour is wet, stop pumping and allow vapour to come out freely, then press and release the button at longer intervals.
After treatment is completed, take out the yellow container, press the button until the pump is empty then close the gas nozzle.

IMPORTANT!
At the end of each treatment, clean the serpentine thoroughly to avoid it gets jammed with any dry product residues.
Take out the liquid tank while Furetto is still on, then keep on pumping as during normal use.
Continue pumping with the device inclined and turned upside down, until no more white smoke is seen.

2.1 MAINTENANCE

No special maintenance is required. Only, dismount the device and clean the sprayer (3).
Spare parts are supplied by and found at our retailers; each part has its code.
Store the device when it has cooled down, in a ventilated area and away from the reach of children.

2.2 PRODUCTS

The substances that can be sprayed with Furetto, are the same that are generally used for acaricide and bactericide treatments, complying with the current rules of the country in which they are used.
The company Industrie Lega 1937 Srl. does not accept any responsibility for using substances not authorized by law.

It is recommended to spray only once and prepare the mixture just before using the device.
In case of a minor infestation, one spray will suffice.
It is advisable to avoid making treatments in a nesting period and to treat at a temperature not below 8 °C.
The treatment should be made once a year, in October or November when supers are inserted.

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- Costruzioni Apistiche, with registered office in Faenza, in Via Maestri del Lavoro 23, supplier of the above-mentioned products, hereby declares **CE conformity** with the following laws:

Directive 2006/42/EC on machinery which amends directive 95/16/EC;

Directive 2014/35/EU on low voltage;

Directive 2014/30/EU on electromagnetic compatibility;

Italian Presidential Decree No. 459 of 24/07/1996 on the harmonization of the laws of Member States relating to machinery.

It also declares that the materials used to manufacture the parts of the above-mentioned products which come into contact with food comply with **food**, hygiene and safety standards, with reference to:

Reg. (EC) 1935/2004 on materials and articles intended to come into contact with food;

Reg. (EC) 2023/2006 on good manufacturing practices;

Italian Ministerial Decree 21/03/1973 hygiene rules for packaging, containers and utensils intended to come into contact with foodstuffs or personal-use products and subsequent amendments and additions;

Italian decree No. 195 of 6/8/2015 amending the decree of the Minister for Health of 21/03/1973 concerning stainless steel;

Reg. (EC) 1895/2005 on epoxy derivatives;

Reg. (EU) 10/2011 on plastic materials and articles intended to come into contact with food;

Italian Legislative Decree No. 29 of 10/02/2017, MOCA;

Circular of the Italian Ministry of Health of 17/07/2017.

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The machinery is guaranteed 24 MONTHS starting from the date of sale.

The guarantee is only valid if, when the machine is collected by our customer care or technical service staff, the owner can produce proof of purchase in the form of a fiscal receipt or invoice.

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The guarantee includes free-of-charge repairing and replacement of any part of the machinery that is found to have manufacturing or material defects by the manufacturer or the manufacturer's authorised person. This guarantee shall not apply to damages caused by negligence, misuse or use not in compliance with the directions contained in the instruction manual, as well as in case of accidents, alteration, tampering, wrong repairing or repairing with non-original parts, repairing by persons not authorised by Industrie Lega 1937 Srl. and damages during transport to/from the purchaser's. All electric parts (electric motors, controls etc.) and parts exposed to normal wear and tear as well as aesthetic parts are also not covered by the guarantee. All labour, packing, forwarding and transport charges shall be borne by the purchaser. Any defective parts which have been replaced shall be retained by and become the property of Industrie Lega 1937 Srl. Any breakdown or defect which should occur during the guarantee period or after its last date shall not in any case entitle the purchaser to suspend the payments nor to any discount off the price of the machine. In any case, Industrie Lega 1937 Srl. shall not be held responsible for any damages resulting from the incorrect use of the machinery.

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